live your life.

Orthoses Instructions and Guidelines

live your life.

We work towards a single goal - to improve function and encourage independence for the patient.

DM Orthotics is a world leading provider of medical rehabilitation and sports performance products.

We provide elastomeric fabric dynamic movement orthoses® to over 25 countries helping to improve the lives of children and adults who face a range of physical challenges.

We are dedicated to the research, development and evolution of our products, with the aim to constantly improve the lives of the people who use them. Everything we create undergoes the highest levels of clinical and scientific testing to ensure that it works at an optimal level for our customers.

Our design and production process incorporates state-of-the-art technology with traditional tailoring, taking the most successful elements from each to produce individually tailored dynamic movement orthoses® that are unmatched in their function and quality.

contents

4
Products

10
English

11
Chinese

12
Czech
Republic

13
Danish

14
French

15
German

16
Hungarian

17
Italian

18
Japanese

19
Norwegian

20
Portuguese

21
South
Korean

21
Spanish

dmo® suit

dmo® glove

dmo® dorsiflex sock

dmo® shorts

dmo® postural scoliosis suit

dmo® single shoulder orthosis

english

wear and washing instructions

Follow the care instructions carefully. By taking care of your orthosis you will prolong its wear and prevent damage.

DO

- > Put the orthosis on and off as advised by your therapist/supplier (see website for more details).
- > Take care when putting it on and taking it off – pulling it on and off roughly and long finger nails can cause damage.
- > It is important to build up wear time as per your individual instructions.
- > Follow the washing instructions carefully or it can become damaged.
- > Contact your therapist / supplier **IMMEDIATELY** if it is getting really tight, causing red marks or has developed ladders and /or holes.

WEARING TIME

DO

- > Build up wearing time slowly over the first week, as stated below, doubling the amount of time daily until the orthosis is worn throughout the working day.

e.g. 1st day -	1 hour
2nd day -	2 hours
3rd day -	4 hours
4th day -	8 hours
5th day -	All day and thereafter all day every day.

WASHING INSTRUCTIONS

DO

- > The orthosis should be washed at least every 2-3 days.
- > The orthosis can be hand washed cool or machine washed at 30 degrees.
- > Short spin only or roll up in a dry towel to remove excess moisture.
- > Allow to completely dry overnight by hanging up indoors.

DO NOT

- > Wear all day immediately. Wear needs to be built up to ensure sensory input, biomechanical changes and fit are monitored.

DO NOT

- > **DO NOT** dry in strong direct sunlight or on a radiator.
- > **DO NOT WRING OUT** – it will stretch out of shape and will cause damage to the materials.
- > **DO NOT IRON.**
- > **DO NOT BLEACH.**
- > **DO NOT DRY CLEAN.**
- > **DO NOT TUMBLE DRY** – it will shrink.

Regular and correct care of the orthosis will prevent it from developing odour and keep the material supple.

If your orthosis is silver treated, this will need to be re-applied after 100 washes as the silver ions start to disperse after 50 washes.

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